


Anatomage TABLE

Example Scenario #2

Scenario: The Head and Neck curriculum guide has been selected by an instructor for their students to work through as a homework assignment.

Minimize the Anatomage software from the main menu

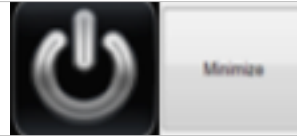


Open the Anatomage Curriculum Folder on the desktop

Open the Head and Neck PDF


This file may also be opened on a separate device or printed for easier access

Review the first few steps in the activity

Go back to the Anatomage software and select the Curriculum button from the main menu

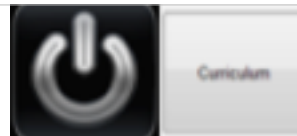

Select the Head and Neck region, Skeletal System, and tap Open

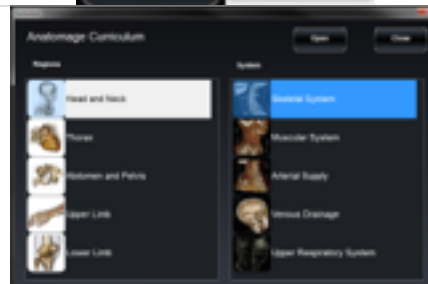

Open the View Sequencer window



Drag the window to the right display for easier viewing

Tap on each listed view in the View Sequencer window to see the associated, interactive image



Compare the images from the PDF to the views in the View Sequencer.

Rather than learning from flat images, the Anatomage Table allows you to interact with each image in a 3D environment